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ENGLISH

1. Wash your hands.
2. Clean and dry anal area well.
3. Remove foil.
4. Release thread from the tampon.
5. Soften the tampon in lukewarm water.
6. Squeeze out the water from the tampon.
7. Place (reusable) applicator in the opening at the bottom of the tampon.
8. Push the tampon past the orbicular muscle, only the thread remains visible.
9. Hold the tampon with the other index finger while withdrawing the applicator. Then push the tampon further inside with your finger.
10. Fasten the thread with sticking plaster.
11. Wash your hands.
12. The tampon can easily be removed by means of the thread. Never wash a tampon through the toilet.

The tampon can be inserted while lying on one's back or side. Compare this procedure to inserting a suppository.

Tips and advice:

1. Do not use the same tampon for more than 8 hours. Build up the duration of use gradually.
2. Before inserting a new tampon, allow some time to empty the bowels.
3. Relax while inserting a tampon.
4. Resistance and pain may occur during first use. Do not force anything.
5. See to good toilet training by going to the toilet at fixed times.
6. Vaseline on the tampon and the anus makes inserting easier. A large applicator will help with a poor hand function.

Contraindications: infections, diarrhoea, prolapse, after radiotherapy.

Always follow the instructions from your specialist or nurse.

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