medi GmbH & Co. KG
 Medicusstraße 1
 D-95448 Bayreuth
 Germany
 T +49 921 912-0
 www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicanada.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 ul. Łąbedzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
 Room 105, Building No. 4,
 Lane 500 ShengXia Road
 Pudong New District
 201210 Shanghai
 People's Republic of China
 T: +86-21 61761988
 F: +86-21 61769607
 info@medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

circaid® juxtafit® premium inelastic compression system

Instructions for use. Gebrauchsanweisung. Инструкция по использованию.
 Instruções de uso. Instrucciones de uso. Mode d'emploi. Gebruiksaanwijzing.
 Istruzioni per l'uso. Bruksanvisning. دليل الاستخدام. Brugsvejledning.
 Instrukcja zakładania. הוראות שימוש.

medi. I feel better.


circaid juxtafit premium
 ankle foot wrap


circaid juxtafit premium
 interlocking ankle foot wrap


circaid juxtafit premium
 lower leg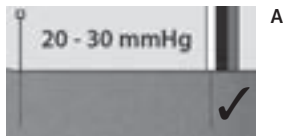

A

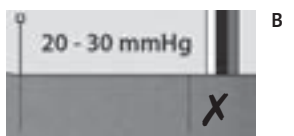B

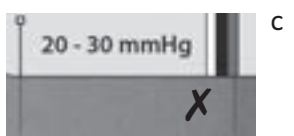C

Lower Legging

Knee Piece

Upper Legging

Whole Legging

Lower Legging with Knee Piece

Upper Legging with Knee Piece


4 049772 365715

EO12044/07/2020


Shelf straps

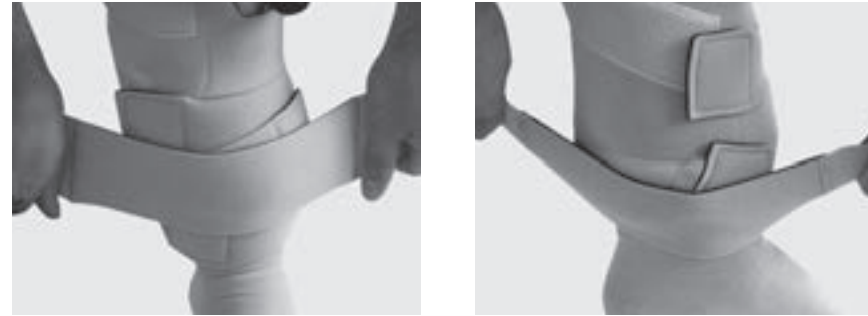Apply knee cover

Built-In-Pressure card

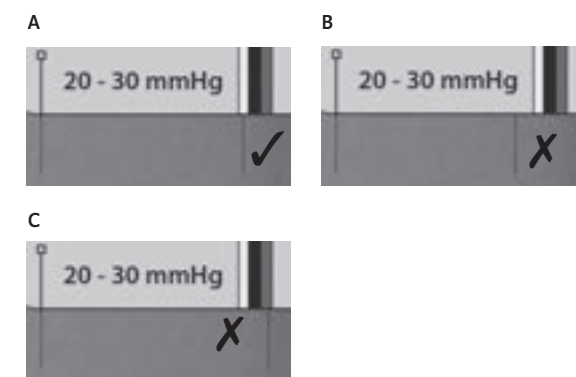circaid® juxtafit® premium

Purpose

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.

The device consists of a series of juxtapositioned, inelastic bands extending from the central portion of the garment.

The wrap contours to the leg and is designed to cover some or all of the leg, depending on the custom design.

Indications

- Lymphedema
- other forms of edema
- Lipedema
- Chronic venous insufficiency
- Varicose veins
- DVT/Thrombosis Prevention
- Post Thrombotic Syndrome
- Venous Stasis Ulcer
- Post Sclerotherapy
- Lipodermatosclerosis

Contraindications

- Severe Peripheral Arterial Disease
- Uncontrolled Congestive Heart Failure
- Septic Phlebitis
- Phlegmasia Cerulea Dolens
- Untreated infection
- Any circumstance where increased venous and lymphatic return is undesirable
- Suspected or known untreated acute DVT

Cautions

- Mild to Moderate Peripheral Arterial Disease
- Impaired Sensation - must be able to sense applied pressure
- Intolerance to material(s)

- Pediatric & assisted use - must be able to sense & communicate applied pressure

Storage instructions

-  Store in a dry place
-  Keep out of sunlight

Lifespan

circaid juxtafit premium has a lifespan of 12 months.

Due to material wear-and-tear the medical efficacy can only be expected for a defined lifespan. This assumes correct handling of the garment (e.g. proper care, application, and removal). 12 month

circaid juxtafit premium shelf strap:
<3 month - depends on the wear and tear

This Package contains

- 1 circaid juxtafit premium compression system
- 1 Built-In-Pressure System™ (BPS™) card
- 1 pair of circaid undersleeves/undersocks
- 1 circaid cover up
- 1 booklet instructions for use

May also include

- 1 foot wrap of choice (lower leg, lower leg with knee, and whole leg only)
- 2 circaid compression anklets
- circaid juxtafit premium shelf straps

Disposal

Please dispose of the garment in household waste. No special criteria for disposal apply for medical compression garments.

medi Ukraine LLC
Evhenia Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Material composition

circaid compression garment
49 % Polyurethane
46 % Nylon
5 % Elastane

For all other components, please refer to the inserts included with each item.

This garment does not contain latex.

Care instructions

The garment can be machine washed using a gentle cycle and dried on a low heat setting. If doing so, it is recommended that you place the garment in a pillow case to aid in keeping the hook and loop tab material lint-free and to protect other clothing from getting damaged.

To extend the life of the garment, hand wash and drip dry.

-
- Machine wash warm
 - Do not bleach
 - Tumble dry, low heat
 - Do not iron
 - Do not dry clean
 - Do not wring
-

Wearing recommendation

Always ensure that the hook tabs are placed on the corresponding landing pads.

Always ensure that the correct prescribed pressure range is being applied.

Slightly loosen the bands of the garment for night-time wear.

Each vertical BPS line on the bands may be at a different location on the leg

depending on the size and shape of the leg. They do not need to be aligned down the front center of the leg or in any other specific location on the leg.

If experiencing any pain, or discomfort, reduce the compression (mmHg) applied or remove device. Inform your clinician of your action.

When bands are placed correctly they should slightly overlap, leaving no space between bands.

Do not tuck bands. The Hook tabs should never touch the skin!

Donning instructions

Step 1: Slide the enclosed circaid undersleeve or undersock onto the leg up to the knee (for lower leggings) or to the groin (for upper leg and whole leg).

Step 2: Position the legging with the black side of the material facing the skin and the seam running along the back of the leg.

Step 3: Unroll the top two bands and loosely secure them to the outside of the legging to anchor it in place. This enables you to easily raise or lower the legging for correct positioning

Note:

If you are using your legging with a circaid foot piece, e. g. pac band or juxtafit ankle foot wraps, follow the application instructions included with your foot piece.

Do not secure the ankle band of the foot piece at this time as it be applied later to overlap the legging.

If using the circaid single band ankle foot wrap, wait until after the legging has been applied to apply the wrap.

Step 4:

Unroll the bottom two bands and secure the bottommost band to the outside of the legging to a firm and comfortable compression level.

The bottom most band can be identified by the care label.

Step 5: While holding the second band, detach the next band. Secure the second band and continue this process with the other bands, going up the leg.

Ensure that the bands are placed in an alternating order.

Step 6: The legging should now lay flat and wrinkle-free against the leg.

Adjust each band to a firm and comfortable compression.

See section “Using the Built-In-Pressure System” for instructions on using the BPS card to set the compression to a prescribed level.

Step 7: Once all the bands are secured, inspect the garment for any gaps or creases. Adjust the bands as necessary. You may now fold the bottom and top parts of the circaid undersleeve or sock over the top and bottom edge of the legging if desired.

Step 8 (optional): Finish applying foot compression. At this time, finish applying your foot compression as directed in the separate instructions. The ankle band will overlap with the lower portion of the juxtafit legging.

Step 9 (optional): Apply shelf straps, if included with your custom compression system. Use the shelf strap to provide additional support or coverage to areas in need, pulling up and around the area on the limb until firm and comfortable compression is achieved.

Step 10 (optional): Apply knee cover, if your compression system covers the knee. Place the knee cover over the knee cap and pull the individual tabs at each corner to firm and comfortable compression. Secure the tabs to the legging once the knee cover is securely against the knee.

Note:

If using a circaid cover up, apply it at this time. Slide the cover up over the legging and secure it using the hook tabs along the top edge. Refer to the instructions for use included with the cover up for additional information.

Doffing instructions

To remove the legging, first remove the cover up, if using. Then remove the knee cover, shelf straps, and/or foot compression, if using any of these. Then unfold the circaid undersleeve.

Detach all bands of the legging starting with the top band. Fold each band onto itself to keep the hook and loop tabs lint-free.

Remove the legging and then remove the circaid undersleeve or sock.

Using the Built-In-Pressure System (BPS)

Step 1: Locate the Built-In-Pressure card in your packaging.

Step 2: Identify your prescribed pressure scale using the color-coded system.

Step 3: Starting with the bottom band, line up the black triangle on the card with one of the BPS lines on the bottom band.

Step 4: Note where the second BPS line lines up with the card’s compression ranges (A).

Step 5: If the BPS line on the garment either falls short (B) or goes beyond the correct compression range (C), readjust the band as necessary so that the second BPS line on the garment is aligned with the prescribed compression range (A). Completely loosen the band before reapplying it if too much tension has been applied.

Step 6: Repeat the steps 3 - 5 for each band going up the leg.
Be sure to adjust all bands to the same tension to ensure gradient compression. Bands may need adjusting throughout the day to maintain the prescribed compression range as edema is reduced.

Note:

Please note it is recommended that the limb is always treated in its entirety with medical compression. This means that compression products for the leg begin at the foot or the ankle and extend up to below the hollow of the knee or beyond to include the thigh.

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